PORTALWEB FISAUDE SL
832403495
Calle Constitución 124
28946 Fuenlabrada, Madrid
91 632 77 40
tienda@fisaude.com

LÁMPARA UV WOOD LAMP

ESPAÑOL

ADVERTENCIA

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ENGLISH

WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.



The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

PACKAGE CONTENTS

• These components are subject to change without prior notice.


• Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	1004 LAMP	-	1
2	SUNSHADE COVER	-	1

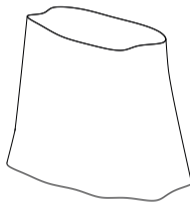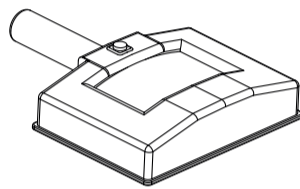INSTRUCTIONS OF USE

1. Hook the sunshade cover from the Wood lamp.

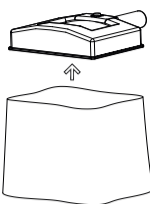2. Hook the other end of the sunshade cover from the customer's body part to be examined.

SKIN CONDITION	DISPLAYED COLOR
Thick epidermis	White fluorescence
Necrosis cells	White spot
Healthy skin	Blue and white
Water deficiency (thin skin)	Purple
Water deficiency	Light purple
Water abundance	Bright fluorescence
Dark fleck	Brown
Oiled part and pimple	Yellow or pink

NOTES

• The treatment should be performed in a dark room.

• Healthy skin on the head and hair under the Wood lamp should appear white and grey-purple.

• Nails should appear bright white.

CAUTIONS

• During the treatment, the customer must close their eyes or put a cotton cloth over their eyes.

• The Wood lamp should not be used for a long time, and the lamp should not touch the skin. The beautician should keep track of the examination time.

• Before using the Wood lamp, the customer must wash their skin. No make-up, beauty cream or beauty liquids are allowed.

TECHNICAL SHEET

Weights and sizes	Net weight (kg)	0,9
	Packing size (cm)	30x21x7
Lamp features	Diopters	3
	Type of luminaire	Ultraviolet
Electrical characteristics	Rated voltage	220V ~ 240V
	Frecuency	50Hz/60Hz
	Rated power	22 W

PORTALWEB FISAUDE SL
B32403495
Calle Constitución 124
28946 Fuenlabrada, Madrid
91 632 77 40
tienda@fisaude.com

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