


Fall Risk- FR

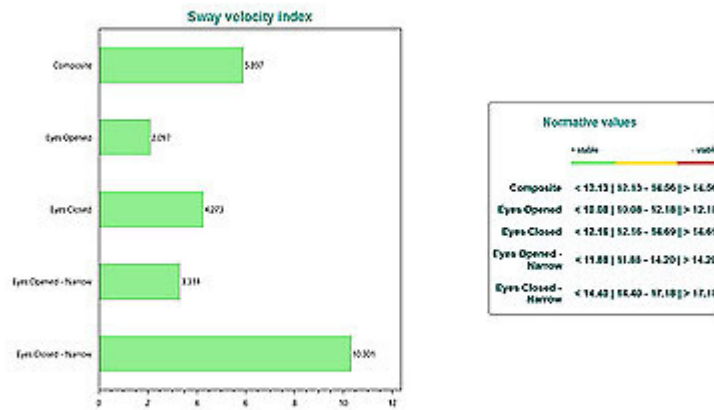

The FR protocol measures the static balance in four conditions:

- 1- Comfortable stance with eyes open;
- 2- Comfortable stance with eyes closed,
- 3- Narrow stance with eyes open
- 4- Narrow stance with eyes closed.

After performing all the conditions of the protocol, the value of the sway velocity index for each of the conditions appears. This sequence of tests can provide a fall risk prediction.

Outputs with normative values comparison: Velocity, Composite velocity, Sway velocity index, Composite SVI, Z-score, Composite Z-score